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EXAFLEX

EXAFLEX High-density collagen matrix, made up of non-reticulated bovine pericardium. It comes from cattle less than 20 months old. The product allows the formation of newly formed collagen and the subsequent vascularization of the implant. Subsequently, the implant will be remodeled and replaced with host tissue. In case of contact with viscera, the EXAFLEX device reduces adherence phenomena to a minimum.

INDICATIONS FOR USE: The EXAFLEX membrane is intended for the following uses:

- Urinary incontinence, pelvic floor reconstruction, vulnology
- Repair of vaginal and rectal prolapse
- Dura mater substitute
- Surgical repair of soft tissue deficiencies, placement of breast implant
- Abdominal and chest wall repair
- Reinforcement of muscle flaps, ligaments and tendons
- Hernia repair, vascular surgery
- Use in dentistry in surgical repair of soft tissue and as a barrier against epithelial invasion of bone tissue.

CONTRAINDICATIONS: Known or presumed sensitivity to derivatives of bovine origin. The product has not been tested on pregnant subjects.

ADVERSE REACTIONS: As with any surgical procedure, adverse reactions are possible.

PRECAUTIONS: The product is disposable, therefore after opening the package, any leftovers cannot be reused or re-sterilized. Before use, check the integrity of the packaging, otherwise do not use. Improper use could cause complications of a surgical nature. Do not use the product if not stored according to the manufacturer's instructions.

ATTENTION: During its use, the product must never be dehydrated, use constant immersions in sterile physiological solution. In case you are forced to use the product in contaminated sites, it is recommended to take appropriate local and systemic measures in order to contain the contamination/infection.

STORAGE: The product must be stored in clean and dry environments. The storage temperature during the shelf life of the product must not be lower than 0°C / 32°F. DO NOT store in refrigerated cells.

TRANSPORT: Transport with intact packaging can take place under normal conditions. Both the product and the original packaging may be subjected to sudden changes in temperature for short periods.

INSTRUCTIONS: The instructions below are not guidelines for surgical technique and are not a substitute for institutional protocols or professional clinical judgments regarding patient care. Check that the product packaging is in good condition and that the packages containing the membrane are intact. Open the packaging respecting the asepsis of the operating room. Rehydrate the membrane by immersion in sterile saline at room temperature. The

product can be soaked even for long periods, 2 or more hours. The product can be immersed in antibiotic-loaded physiological solution. Use the product according to surgical needs.

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